

Hand Held Vascular Doppler model VD8M

A tool used in the assessment of lower limb arterial and venous disease

The hand held Unidirectional Doppler is an instrument that detects blood flow using the principles of the Doppler effect. It can be a very useful tool in assessing how good the blood flow is to the foot or hand. It is a painless test and very similar to taking blood pressure using a stethoscope. It is particularly useful for assessment of arterial disease but can also be used for venous problems.

The VD8M features a hand held Doppler with superior sound quality, extended battery life, enhanced probe sensitivity, interchangeable probes and a one year full warranty. It includes a battery recharging system so you don't have to worry about changing the battery often.

Features:

- ◆ Interchangeable Doppler Probes
- ◆ Unidirectional Doppler
- ◆ 8MHz Vascular Probe is Standard
- ◆ Continuous Wave Doppler
- ◆ Dry-cell or Rechargeable battery operation
- ◆ Charger supplied free

Specifications:

Dimensions (H,W,D)	: 125 x 75 x 26mm
Weight	: 190 grams
Battery	: 2 x 1.2V Rechargeable Batteries
Audio bandwidth	: 350Hz to 2KHz, 0.5 W

Applications:

- ◆ Ankle Brachial Index Study
- ◆ Arterial and Venous examination
- ◆ Carotid Bifurcation
- ◆ Penile artery study




Diabetic Foot Care India Pvt Limited

No.18/1, Kannappanagar, 3rd Main Road, Thiruvanmiyur, Chennai - 600 041
 Tel - 91-44-43564129/32514129 E-mail : mesmedi@gmail.com

ID : 270820131513
 Name : Mr.Gopalakrishnan
 Age : 54 Yrs

Gender : MALE
 Date : 27/Aug/2013
 Examiner :

Ankle Brachial Index Report

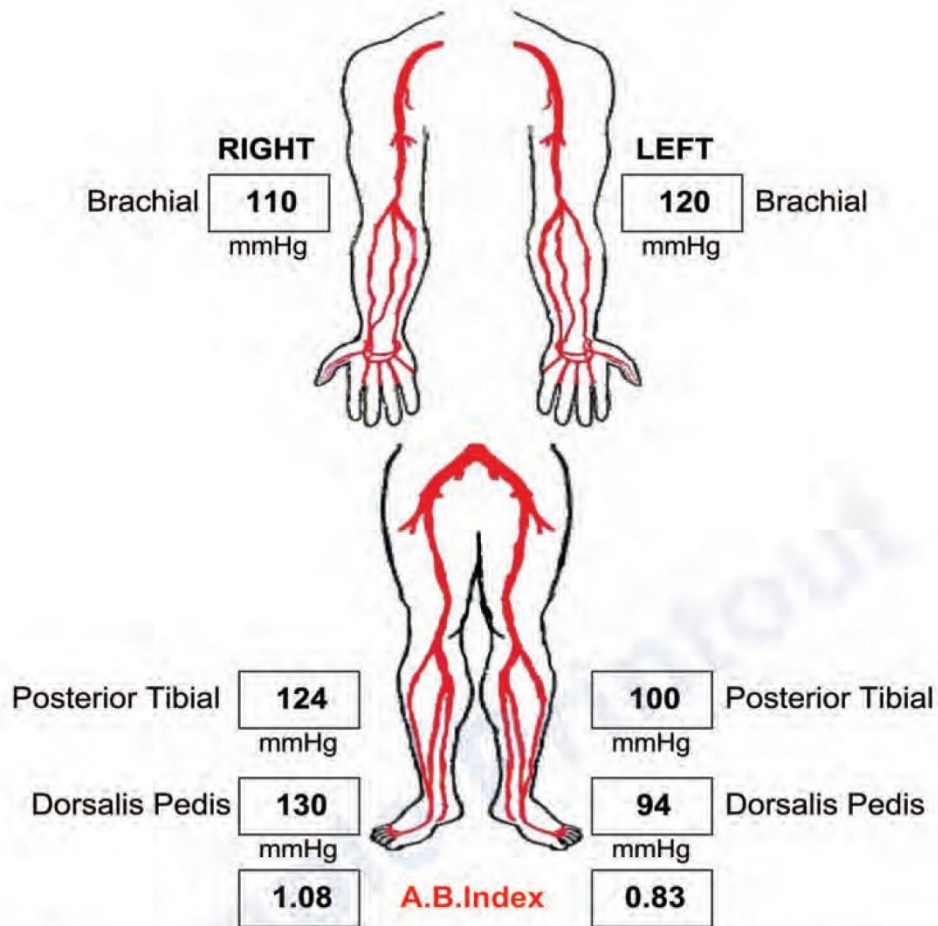

Impression :

Normal right arterial study
 Mild left arterial disease

Dr. Rajesh Kumar
 Vascular Surgeon

TASC II Guideline for ABI

0.91 to 1.30	- NORMAL
0.71 to 0.90	- MILD PAD
0.41 to 0.70	- MODERATE PAD
< 0.40	- SEVERE PAD
> 1.30	- INCOMPRESSIBLE (CALCIFIED) ARTERY

Diabetic Foot Care India Pvt Ltd, Chennai. Ph:+91-44-32514129 / 43564129; +91-9380621607; E-mail:mesmedi@gmail.com

For more information contact



Diabetic Foot Care India Pvt Limited

No.18/1, Kannappanagar, 3rd Main Road, Thiruvanmiyur, Chennai-41, INDIA.
 Tel - 91-44-4356 4129 / 4359 0510 / 93806 21607
 E-mail: mesmedi@gmail.com; elango@mesmedi.com
 URL: www.diabeticfootcareindia.com; www.diabeticneuropathy.co.in

